



# MASSAD® IMPRESSION TRAYS


Massad impression trays are made of a low temperature thermoplastic material, making them limber and moldable when placed into a hot water bath. The heated tray can be placed in the mouth and molded intraorally using patient facial muscles or finger pressure. Because the tray material has no memory, it retains the heat molded position. Massad impression trays are available in three styles.

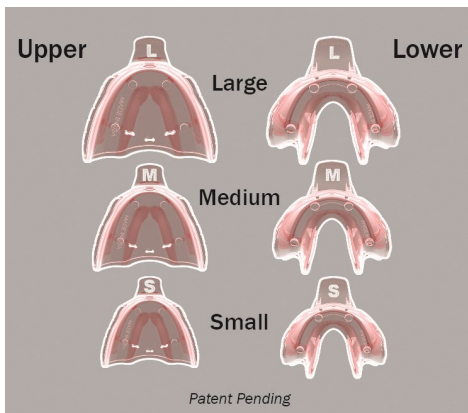

Massad® Overdenture Low Temp Tray

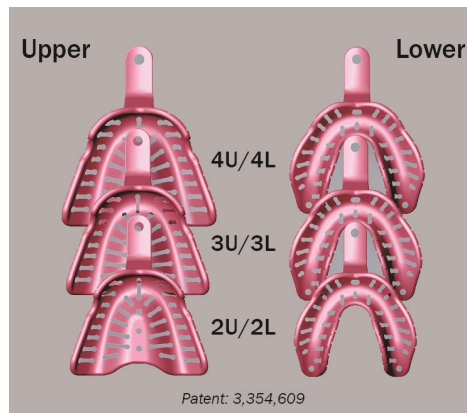

Massad® Edentulous Low Temp Tray

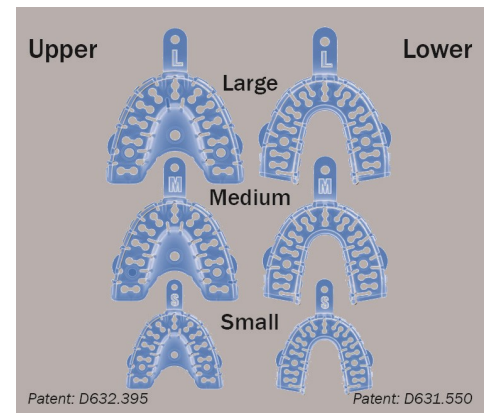

Strong Massad® DenPlant™ Low Temp Tray

## Massad Overdenture Low Temp Tray - Patent Pending

Specially designed for overdentures and selective partial dentures. Due to the tray's color and clarity, practitioners can accurately see tray placement in the patient's mouth, with or without material, which results in a great impression the first time, saving time and material that would be used if the impression needed to be repeated. The handle is uniquely shaped to have zero interference with performing an accurate border molding.

- Available in three upper sizes (small, medium, large) and three lower sizes (small, medium, large), to ensure the perfect fit for any patient. Set of twelve trays includes two of all sizes.
- Color: translucent pink

## Massad Edentulous Low Temp Tray - Patented

Specifically designed for edentulous and implant patients. It delivers incredibly accurate impressions while maintaining productivity. Retention slots allow for proper flow of impression material. It was created to combine both the preliminary and final impressions into one step by achieving optimum results. The handle was engineered to avoid interference from the opposing lip tissue during the impression, and it feels extremely comfortable and balanced in the hand.

- Available in three upper sizes (2U, 3U, 4U) and three lower sizes (2L, 3L, 4L). Set of twelve trays includes two of all sizes.
- Color: opaque pink

## Strong Massad DenPlant Low Temp Tray - Patented

For partials, immediates, implants, crowns & bridges, and orthodontic models. It provides better clarity, strength and design than conventional trays, and for a variety of different procedures. It is the most versatile and feature packed tray on the market. The tray's crystal clear material allows you to view through the tray and visualize the void allowed for your impression material, including a physiological shape that allows you to easily maintain functional anatomy, an extended handle for ergonomics, and retention slots for proper flow of material. Key features include a deep and wide tooth channel that will accommodate most dental arrangements.

- Available in three upper sizes (small, medium, large) and three lower sizes (small, medium, large). Set of twelve trays includes two of all sizes.
- Color: translucent blue



## RANSOM & RANDOLPH

3535 Briarfield Boulevard | Maumee, OH 43537 USA  
800.800.7496 | 419.865.9497 | 419.865.9997 (FAX) | [www.ransom-randolph.com](http://www.ransom-randolph.com)



Investing with Innovation™

---

# MASSAD® IMPRESSION TRAYS

---

## Application Instructions

1. Place the tray into a hot water bath for approximately 30 seconds at 165 degrees.
2. When the tray is easily manipulated and becomes rubbery, it is quickly inserted into the mouth.
3. Once the heated tray is placed in the mouth, it can be molded intraorally using patient facial muscles or finger pressure.
4. For an accurate impression, the goal is to have a minimum of one millimeter of impression material in all dimensions.

*As the conditions or methods of use are beyond our control, we do not assume any responsibility and expressly disclaim any liability for any use of this product. Information contained herein is believed to be true and accurate but all statements or suggestions are made without warranty, expressed or implied, regarding accuracy of the information, the hazards connected with the use of the material or the results to be obtained from the use thereof. Compliance with all applicable federal, state, and local regulations remains the responsibility of the user. All potential liability related to the sale and use of this product is limited to the cost of the particular goods sold in their respective transactions.*



**RANSOM & RANDOLPH**

3535 Briarfield Boulevard | Maumee, OH 43537 USA  
800.800.7496 | 419.865.9497 | 419.865.9997 (FAX) | [www.ransom-randolph.com](http://www.ransom-randolph.com)



Issue Date: August 17, 2023

*Investing with Innovation™*