

# OXYPLUS™ INVESTMENT & LIQUID



Oxyplus investment is an oxyphosphate dental investment for making refractory models for waxing and casting partial frames.

When making the refractory model, Oxyplus liquid is recommended at a diluted ratio of 3 parts deionized or distilled water to 1 part Oxyplus liquid by volume. Oxyplus liquid ratios will need to be slightly adjusted based on your geographical location and/or climate conditions.

Please note:

- More Oxyplus liquid = more expansion for looser fitted castings
- Less Oxyplus liquid = less expansion for tighter fitted castings

**When investing the outer mold, Oxyplus investment is mixed with deionized or distilled water only. Do not use Oxyplus liquid.**

## Duplicating the Gypsum Master Model

1. Soak the master model in water for 30 minutes (or 2 minutes under vacuum) to remove any air from the gypsum model.
2. Blow off any excess water from the model and place the model into a duplicating flask.
3. Pour in Uniflex™, Perfex®, or Neoflex™ duplicating material from your hydrocolloid conditioning unit.
4. Once filled, place the duplicating flask into a chilled water bath, covering only the bottom of the flask and not the sides.
5. As the duplicating material starts to cool and set up, it will pull toward the cold water, which will cause it to pull tightly over the master model for an exact duplication.
6. Let set for 30 minutes, then remove the master model and allow the duplicating material to bench set to reach room temperature before pouring in Oxyplus investment.

## Making the Oxyplus Refractory Model

	Mechanical Mix	Hand Mix
<b>Powder</b>	200g	200g
<b>Diluted Oxyplus Liquid (3 to 1 Ratio)</b>	25 ml (18.75 ml distilled water to 6.25 ml Oxyplus liquid)	25 ml (18.75 ml distilled water to 6.25 ml Oxyplus liquid)
<b>Mix Time</b>	60 seconds under vacuum	90 seconds

1. Place impression on vibrator and fill the impression with the mixture while vibrating.
2. Remove from vibrator and allow to harden for 30 minutes.
3. Remove the model, trim and dry for 45 minutes at 375-425 °F (190-218 °C).
4. Follow the application instructions for an R&R Rosin Dip and dip the model.

## Investing the Model (Outer Mold)

	Mechanical Mix	Hand Mix
<b>Powder</b>	600g	600g
<b>Deionized or Distilled Water</b>	70 ml	73 ml
<b>Mix Time</b>	60 seconds under vacuum	90 seconds

1. Place the ring on the vibrator and pour the mixed investment into the ring until the wax-up is covered.
2. Vibrate the mix back out of the ring until there is a bubble-free thin coating covering the wax-up.



**RANSOM & RANDOLPH**

3535 Briarfield Boulevard, PO Box 1570 | Maumee, OH 43537 USA  
800.253.4502 | 419.865.9497 | 419.865.9997 (FAX) | [www.ransom-randolph.com](http://www.ransom-randolph.com)




*Investing with Innovation™*

# OXYPLUS™ INVESTMENT & LIQUID

3. Once a bubble-free thin coating is achieved, vibrate the mix back into the ring until filled.
4. Remove immediately from vibrator.
5. Bench set for 60 minutes.

## Burnout

From	To	In	Hold	Raise To	In	Hold
70 °F (21 °C)	550 °F (290 °C)	20 min	30 min	1800 °F (982 °C)	60 min	60 min

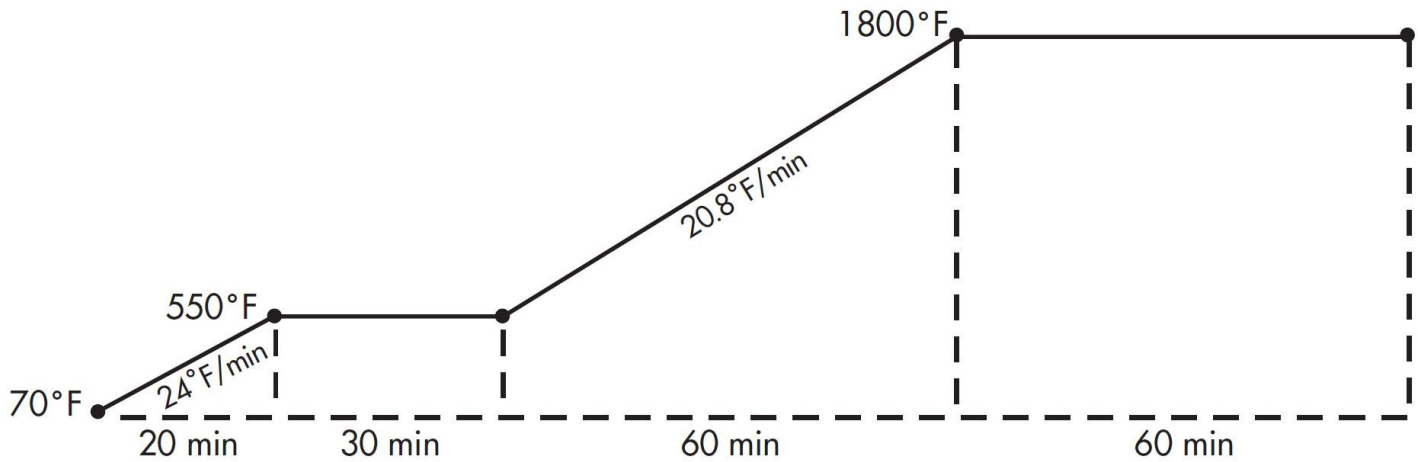

## Storage

Keep container closed when not in use. Store in a cool dry area.

## Technical Tips

For additional information and recommendations, refer to the Dental Investment Defect Analysis Guide Technical Tip available for download at [www.ransom-randolph.com](http://www.ransom-randolph.com).

### **Danger** Crystalline Silica (Quartz and Cristobalite)

May cause cancer by inhalation. Causes damage to lung through prolonged or repeated exposure by inhalation. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Do not breathe dust. Wash thoroughly after handling. Do not eat, drink or smoke when using this product. Use personal protective equipment as required. IF exposed or concerned: Get medical attention. Store locked up. Dispose of contents/container in accordance with local and national regulations.

*As the conditions or methods of use are beyond our control, we do not assume any responsibility and expressly disclaim any liability for any use of this product. Information contained herein is believed to be true and accurate but all statements or suggestions are made without warranty, expressed or implied, regarding accuracy of the information, the hazards connected with the use of the material or the results to be obtained from the use thereof. Compliance with all applicable federal, state, and local regulations remains the responsibility of the user. All potential liability related to the sale and use of this product is limited to the cost of the particular goods sold in their respective transactions.*



**RANSOM & RANDOLPH**

3535 Briarfield Boulevard, PO Box 1570 | Maumee, OH 43537 USA  
800.253.4502 | 419.865.9497 | 419.865.9997 (FAX) | [www.ransom-randolph.com](http://www.ransom-randolph.com)



Issue Date: January 12, 2022

*Investing with Innovation™*