

# UNI-ROCK™ DENTAL STONE



## Type III Dental Stone

An extra smooth, low expanding stone for all types of models.

### Typical Material Properties\*

Type	III
Color	Buff
Water/Powder Ratio	30/100
Set Time	8½ - 10½ minutes
Set Expansion	≤ 0.14%
Compressive Strength (at 1 hour)	> 3,000 psi
Ultimate Compressive Strength	8,500 psi

\*These results are based on the testing methods, frequency and procedures of Ransom & Randolph or its approved suppliers. The levels referenced herein are only for general guidance and do not constitute a firm specification.

### Application Instructions

#### Manual Mixing

1. Add 100 g of powder to 30 ml of water.
2. Allow powder to soak in water for 20 seconds.
3. Hand spatulate at 120 rpm for 60 seconds.

#### Mechanical Mixing

1. Add 100 g of powder to 30 ml of water.
2. Allow powder to soak in water for 20 seconds.
3. Mechanically mix for 20 to 30 seconds at 400 rpm.

### Storage

Gypsum materials are subject to deterioration when exposed to the atmosphere. Store this product at 59-86°F (15-30°C). Keep container tightly closed once seal is broken to protect contents from moisture and high humidity.

### Safety

Danger. Contains crystalline silica. May cause cancer inhalation. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Wear protective gloves/protective clothing/eye protection/face protection. If feeling unwell, get medical advice/attention. See SDS for more information.

*As the conditions or methods of use are beyond our control, we do not assume any responsibility and expressly disclaim any liability for any use of this product. Information contained herein is believed to be true and accurate but all statements or suggestions are made without warranty, expressed or implied, regarding accuracy of the information, the hazards connected*



## RANSOM & RANDOLPH

3535 Briarfield Boulevard | Maumee, OH 43537 USA  
800.800.7496 | 419.865.9497 | 419.865.9997 (FAX)  
www.ransom-randolph.com



Made in the USA

Ransom & Randolph GmbH  
Leipziger Straße 40 | 04571 Rötha Germany  
+49 342 06373999

Investing with Innovation™

---

# UNI-ROCK™ DENTAL STONE

---

*with the use of the material or the results to be obtained from the use thereof. Compliance with all applicable federal, state, and local regulations remains the responsibility of the user. All potential liability related to the sale and use of this product is limited to the cost of the particular goods sold in their respective transactions.*



---

## RANSOM & RANDOLPH

3535 Briarfield Boulevard | Maumee, OH 43537 USA  
800.800.7496 | 419.865.9497 | 419.865.9997 (FAX)  
[www.ransom-randolph.com](http://www.ransom-randolph.com)



Made in the USA

*Issue Date: February 15, 2024*

Ransom & Randolph GmbH  
Leipziger Straße 40 | 04571 Rötha Germany  
+49 342 06373999

*Investing with Innovation™*